

TECHNICAL SERVICE BULLETIN



Technical Service Bulletin #05-004

Date: July 22, 2005
Re: VALIDATE[®] UC1 and UC2
P/N: 701 and 702
Subject: Direct ISE method incompatibility

Maine Standards Company VALIDATE[®] UC1 (reorder number 701) and UC2 (reorder number 702) calibration verification / linearity test sets are not intended for use with Direct ISE methods. The formulation pH will cause problems if used on the Direct ISE method.

UC1 and UC2 should be used for Indirect ISE measurement only. ***DO NOT USE FOR DIRECT ISE MEASUREMENT.***

Please refer any questions related to this bulletin to our Technical Support department.

Thank you,

Technical Support Department
Maine Standards Company
765 Roosevelt Trail
Windham, ME 04062
1-207-892-1300
1-800-377-9684
Fax: 1-207-892-2266
E-mail: info@mainestandards.com