



TECHNICAL SERVICE BULLETIN 09-006

Date Issued: December 11, 2009

Product:

VALIDATE[®] Product	REF
GC1	1100

Subject: Preparation of Calibration Verification / Linearity Test Kits Prior to Use

In order to ensure optimal product performance, it is important to follow the PREPARATION Section of the package insert. All VALIDATE[®] calibration verification / linearity test kits should be brought to room temperature **for a minimum of 45 minutes** and bottles should be **inverted gently several times** before dispensing.

To maximize stability, it is recommended that exposure to room air be minimized. Tightly cap opened bottles and return -10 to -25°C immediately after dispensing.

If you have any questions regarding this bulletin, please call 1-800-377-9684 and ask for Technical Support.

Contact Information:

Technical Support Department
Maine Standards Company
765 Roosevelt Trail
Windham, ME 04062
Phone: 1-800-377-9684
Fax: 1-207-892-2266
E-mail: Info@mainestandards.com