TECHNICAL SERVICE BULLETIN 19-007





Date Issued: March 1, 2019

Subject: Possible 1053 Error Code on A-ALT results when using VALIDATE® GC3ab

(1300ab) with Abbott Architect A-ALT reagent, REF 8L92

Explanation: It has been reported to LGC Maine Standards that when testing ALT in the

VALIDATE® GC3ab test kit (order number 1300ab) using Abbott Architect A-ALT reagent (REF 8L92), Level 6 from the VALIDATE® GC3ab test kit may result in a 1053 Error Code and produce no results for that sample. The 1053

Error Code indicates a rate reaction linearity failure.

Additional Quality Assurance testing performed at LGC Maine Standards confirmed the 1053 Error Code when using Abbott Architect A-ALT reagent. VALIDATE® GC3ab Levels 1 – Level 5 perform as expected with no 1053

Error Code.

If you are using Abbott Architect A-ALT reagent and receive a 1053 Error Code when testing VALIDATE® GC3ab, please contact LGC Maine Standards'

Technical Support.

LGC Maine Standards is evaluating the cause of the 1053 Error Code when

using VALIDATE® GC3ab and diligently working to resolve the issue.

Please note: This 1053 Error Code does not occur when testing ALT in the VALIDATE®

GC3ab test kit using Abbott Architect ALT reagent (REF 7D56-21).

Contact Information: Technical Support Department

Maine Standards Company, LLC

221 US Route 1

Cumberland Foreside, ME 04110 Phone: 1-800-377-9684 ext. 1428

+1-207-892-1300 ext. 1428

Fax: +1-207-892-2266

E-mail: msc.techsupport@LGCGroup.com